


Accessories

IN THIS SECTION

Backpack and Seat Pouch (Quickie)	11.1	Positioning Belts.....	11.6
Catch-alls	11.2	Positioning Belt (Quickie Mach 2).....	11.6
Crutch Pouch	11.2	Side Guards (Quickie 3D/Kidz Bike/Mach 2).....	11.7
Cyclometers	11.3	Plastic Side Guard (Quickie All Court).....	11.8
Fender (Quickie Cycl-one) (Disc.).....	11.3	Side Guards (Quickie Heat).....	11.8
Fender (Cyclone)	11.4	Spoke Guards	11.9
Fender (Quickie Mach 2)	11.4	Spoke Guards (Sport)	11.10
Floor Pump.....	11.5	Spoke Guard (Sport) Reference Guide.....	11.10
Friction Tape.....	11.5	Tool Kit (Quickie Cyclone).....	11.11
Glue.....	11.5	Tool Kit (Quickie Mach 2)	11.11
Helmet.....	11.5	Tow Bar (Quickie Mach 2).....	11.12
Impact Guards.....	11.6	Water Bottle	
Leg Straps (Quickie All Court/Matchpoint)	11.6	(Quickie Cyclone/Kidz Bike/Mach 2).....	11.12


BACKPACK AND SEAT POUCH (QUICKIE)

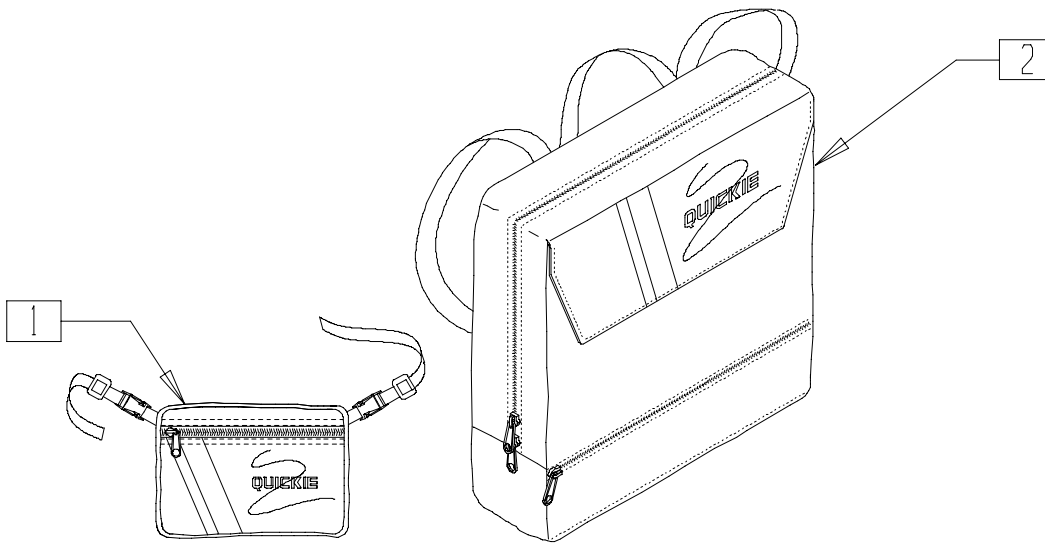

SHAC0X01

ITEM	NUMBER	DESCRIPTION	QTY
1	750304-750306	SEAT POUCH	1
2	750120-750125	BACKPACK	1

CATCH-ALLS

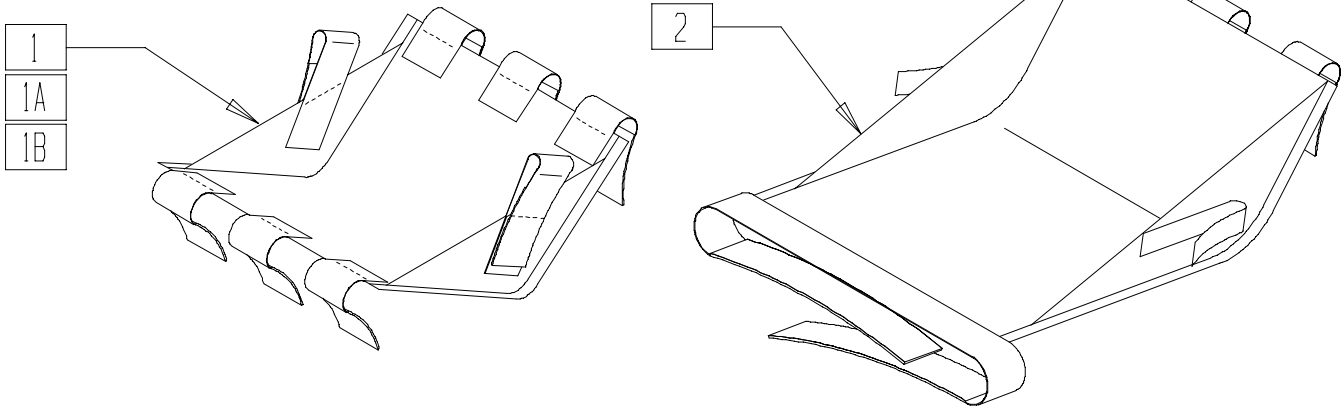


AC0X029

ITEM	NUMBER	DESCRIPTION	QTY
1	750340	CATCH-ALL SMALL 12"-16"	1
1A	750341	CATCH-ALL LARGE 17"-20"	1
1B	750337	CATCH-ALL QUICKIE 1 AND 2 SMALL	1

ITEM	NUMBER	DESCRIPTION	QTY
2	750338	CATCH-ALL DT 85°/GP-GPV SMALL	1
	750336	CATCH-ALL DT 85°/GP-GPV LARGE	


CRUTCH POUCH

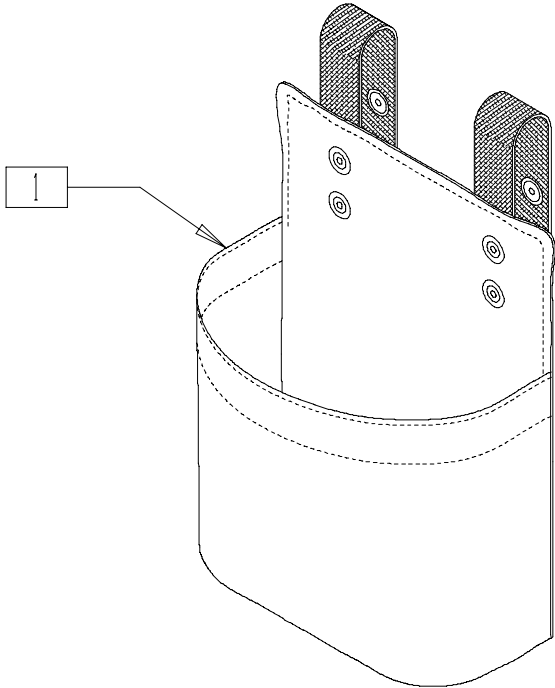

AC0X043

ITEM	NUMBER	DESCRIPTION	QTY
1	750323	CRUTCH POUCH	1

CYCLOMETERS

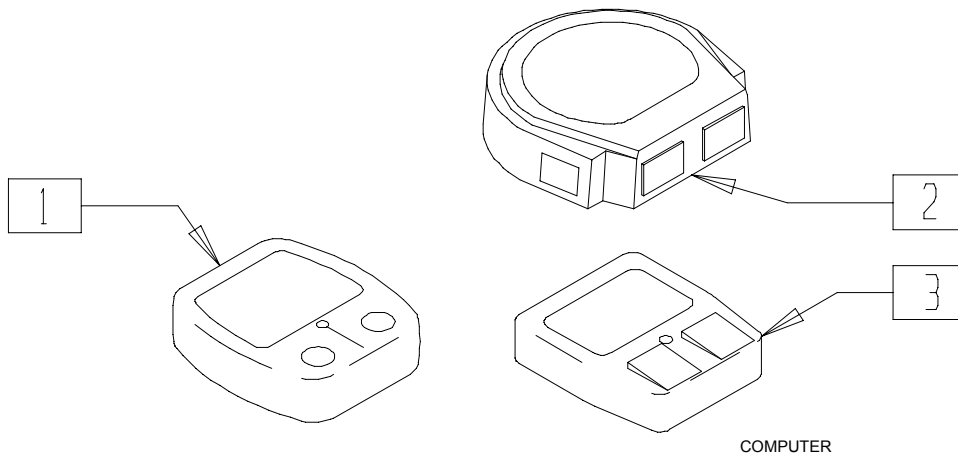


COMPUTER

ITEM	NUMBER	DESCRIPTION	QTY
1	444100	COMP MULTIFUNCTION W/CADENCE (CYCL-ONE, MACH 2, MACH 3)	1
2	444104	COMP CATEYE CORDLESS 2 (RACER)	1
3	443032	COMP CATEYE MITTY 2 (RACER)	1

FENDER (QUICKIE CYCL-ONE)

(Discontinued 2/1/00 - parts only)

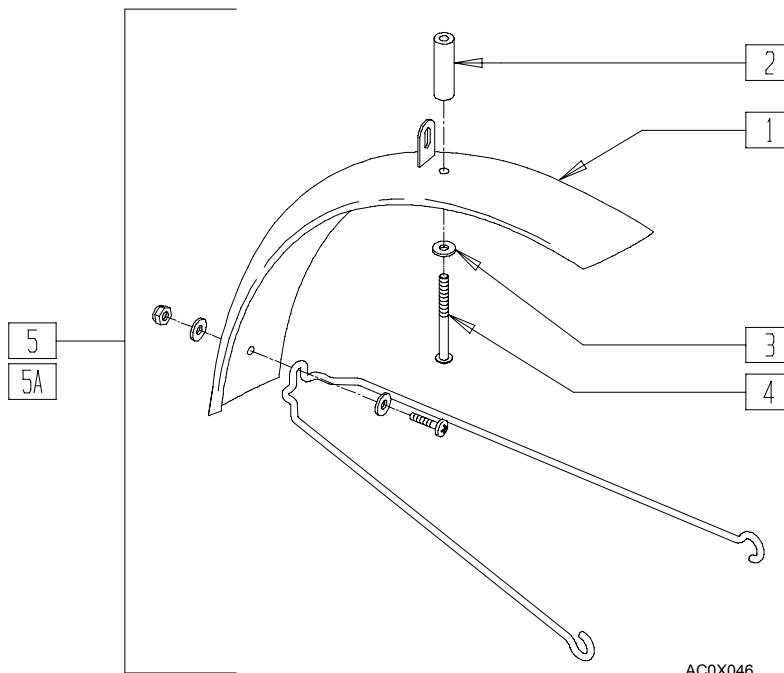


AC0X046

ITEM	NUMBER	DESCRIPTION	QTY
1	444097	FENDER 20" CHROME 44mm	1
2	561038	HANDRIM SPACER (LONG)	1
3	262099	3/16 ALUMINUM WASHER BACKUP	1
4	281175	10-32 x 2 1/8 PAN PHIL SS	1
5	444096	FENDER 20" CHROME W/SPACER	
5A	444097*	FENDER 20" CHROME 44MM	

NOTE: ORDER 444097 WHEN USING SECOND HAND BRAKE.


FENDER (CYCLONE)

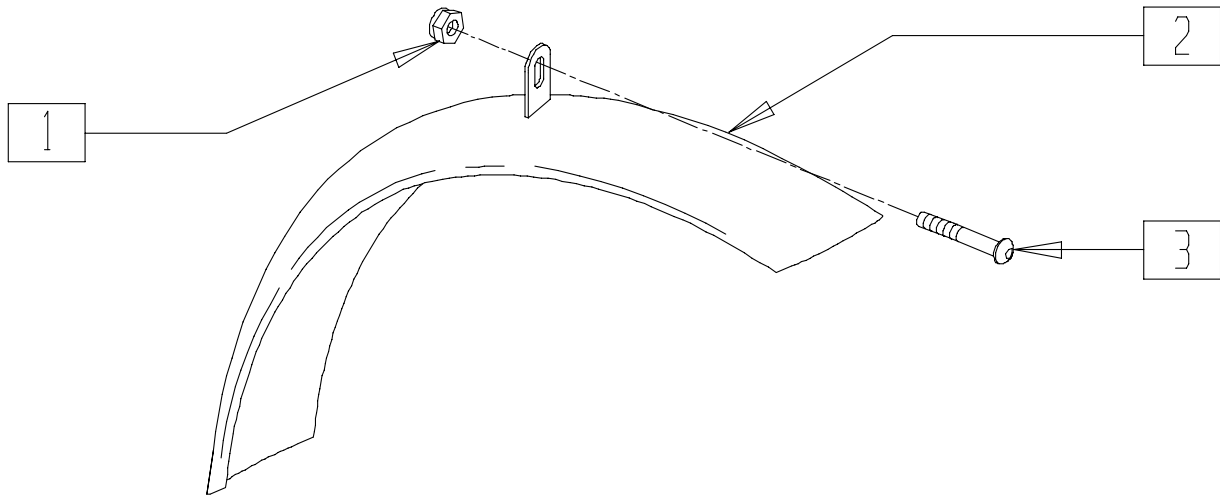

CYC00X07

ITEM	NUMBER	DESCRIPTION	QTY
1	201110	1/4-28 NYLOCK NUT	1
2	444098	FENDER SPLASH GUARD	1
3	226129	1/4-28 x 1 1/4 BH C/S	1


FENDER (QUICKIE MACH 2)

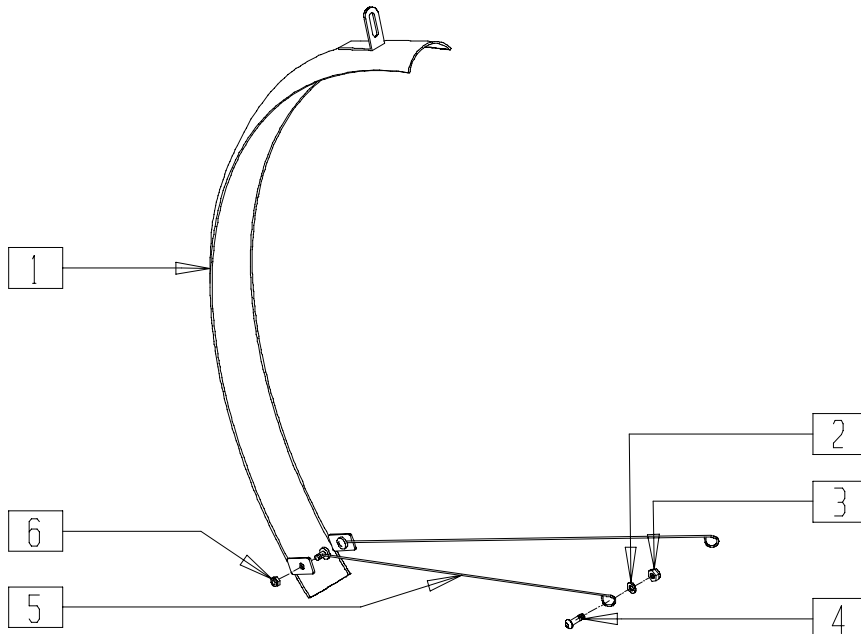

LC00X22

ITEM	NUMBER	DESCRIPTION	QTY
2	262099	3/16 ALUM WASHER BACKUP	2
3	200101	10-32 NYLOCK NUT THIN	1
4	226211	10-32 x 1/2 HEX C/S SS	1
1,5,6	442064	FENDER FRONT FROM REC 2634	


FLOOR PUMP

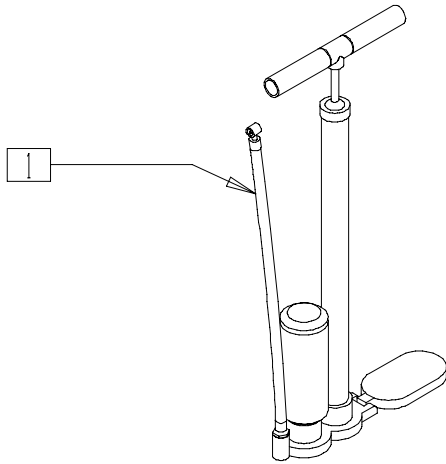

LC00X21

ITEM	NUMBER	DESCRIPTION	QTY
1	389050	TIRE PUMP	1


FRICTION TAPE

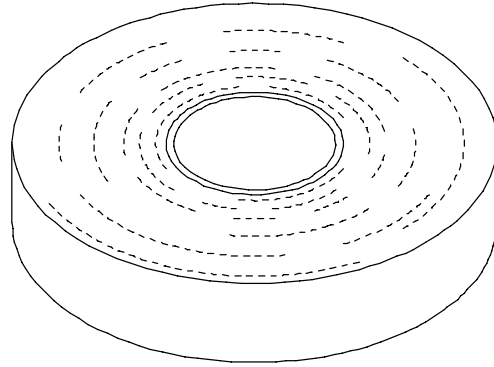

AC0X049

ITEM	NUMBER	DESCRIPTION	QTY
1	160236	TAPE FRICTION	1


GLUE

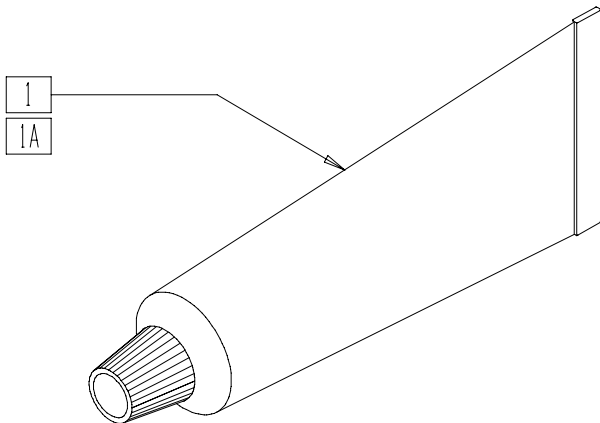

AC0X047

ITEM	NUMBER	DESCRIPTION	QTY
1	160227	HANDBALL GLUE HR	1
1A	381351	QP TUBULAR TIRE GLUE	1


HELMET

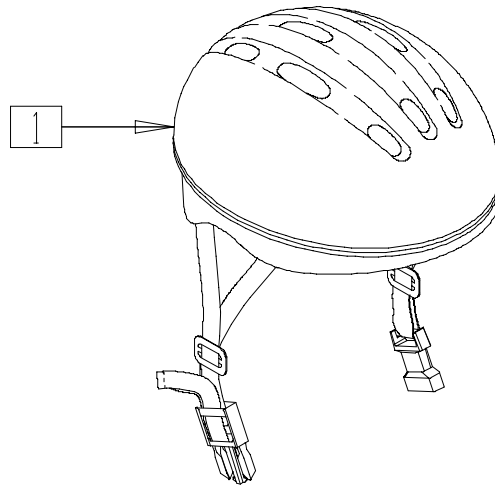

AC0X044

ITEM	NUMBER	DESCRIPTION	QTY
1	440072	HELMET BREEZE MED BLK	1
	440070	HELMET BREEZE LG BLK	
	440092	HELMET MED/LG WHITE	
	440090	HELMET LG/XLG WHITE	


IMPACT GUARDS

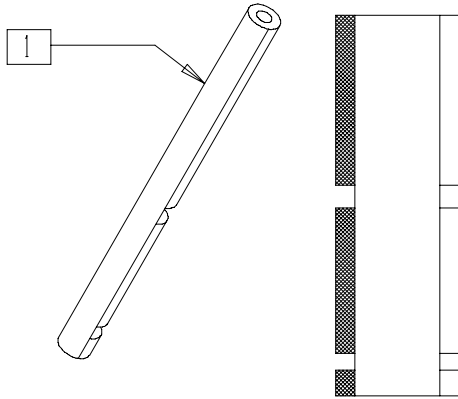

AC0X039

ITEM	NUMBER	DESCRIPTION	QTY
1	750324	IMPACT COVER GUARDS SM (PR)	1
	750325	IMPACT COVER GUARDS STD (PR)	
	750326	IMPACT COVER GUARDS LRG (PR)	
	580060	IMPACT GUARD VINYL TUBING 24"	


LEG STRAPS (QUICKIE ALL COURT/ MATCHPOINT)

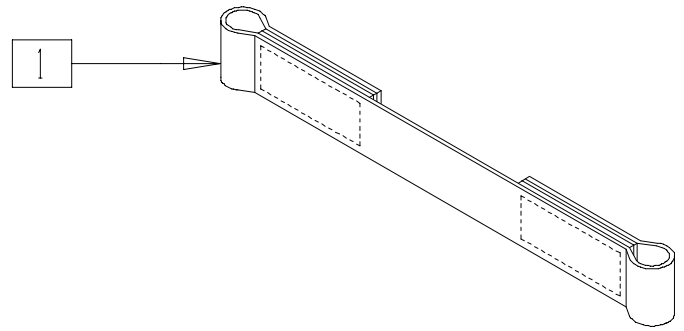

FT0X015

ITEM	NUMBER	DESCRIPTION	QTY
1	740111-740132	LEG STRAP 10"-20" (EA)	1


POSITIONING BELTS

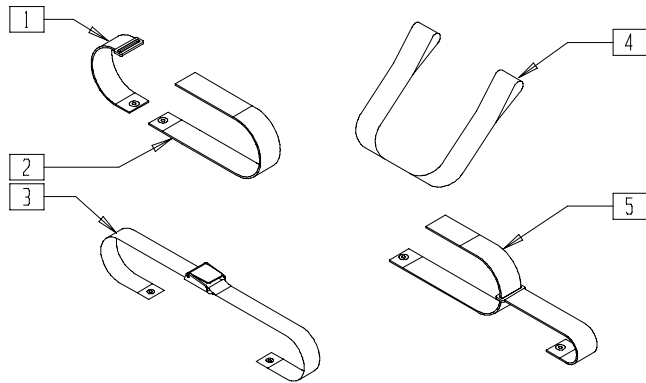

RBY0X09A

ITEM	NUMBER	DESCRIPTION	QTY
1	750220	LEG STRAP W/BUCKLE® 3D	1
2	750221	LEG STRAP W/VELCRO® 3D	1
1,2	740222	LEG STRAP ASSM 3D	
3	751025	POSITION BELT NYLON BKL (SPECIFY DIMENSIONS)	1
4	740068	LONG STRAP (QR)	1
5	751020	POSITIONING BELT HK/LP CLS (SPECIFY DIMENSIONS) (QUAD RUGBY/RACER)	1


POSITIONING BELT (QUICKIE MACH 2)

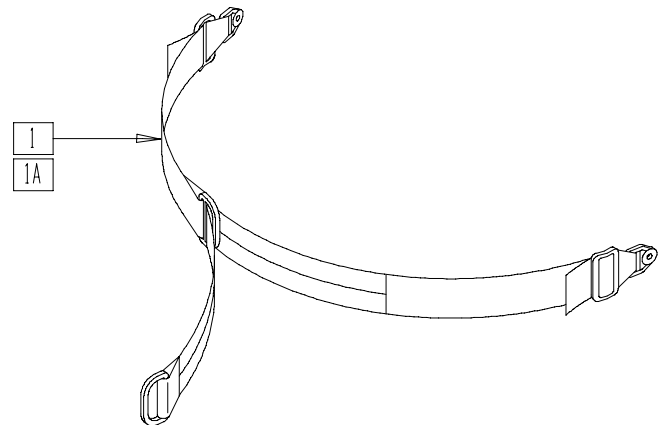

LC00X19

ITEM	NUMBER	DESCRIPTION	QTY
1	450410	CZ BELT 2" W/VELCRO® 34"	1
1A	450413	CZ BELT 2" W/VELCRO® 40"	1

SIDE GUARDS (QUICKIE 3D/KIDZ BIKE/MACH 2)

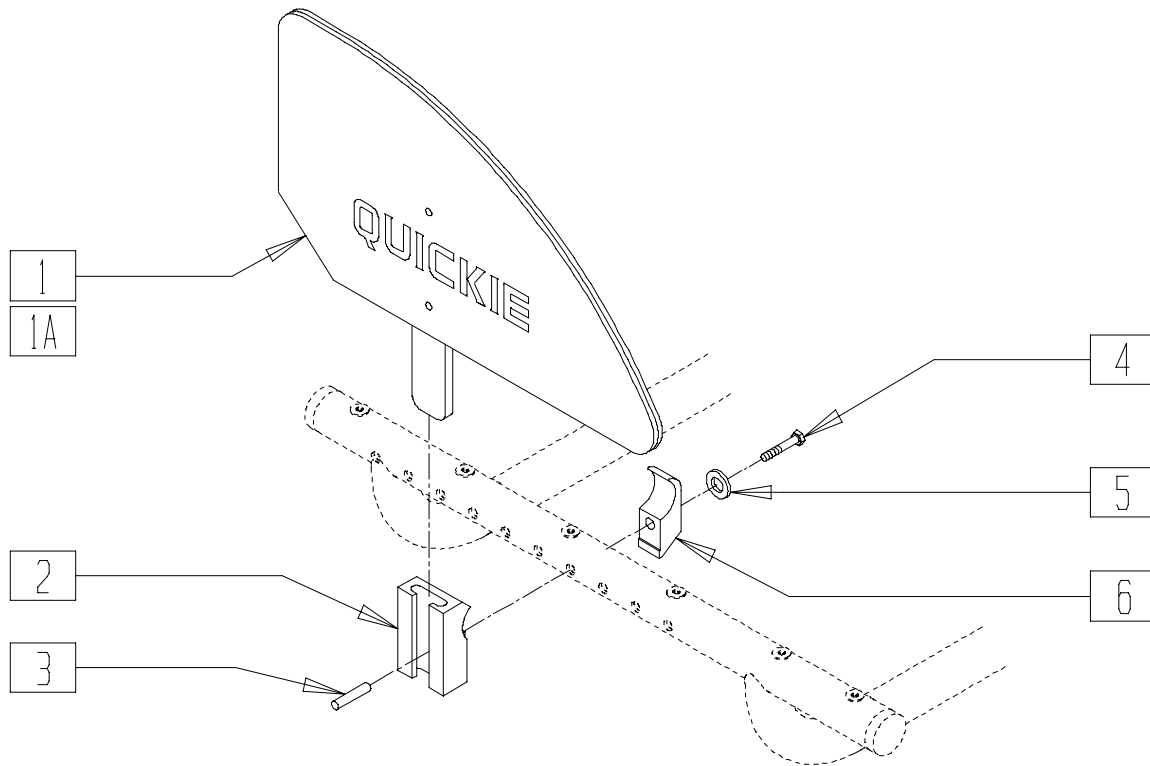


LC00X11


ITEM	NUMBER	DESCRIPTION	QTY
1	881150, 881151	PLASTIC SIDE GUARD KDS (EA)	1
1A	881152, 881153	PLASTIC SIDE GUARD STD (EA)	1
2	550327	SIDE GUARD REC W/HOLE (M2/M3)	1
	550325	SIDE GUARD/KIDZ ARM REC (3D)	
3	483093	SIDE GUARD PIN M3/M2	1
	341130	1/4 x 1 1/8 ROLL PIN SS	
4	221127	1/4-28 x 1 3/8 HEX C/S	

ITEM	NUMBER	DESCRIPTION	QTY
5	262110	1/4 AN WASHER FLAT THIN	1
6	550251	CLAMP P300 CHRNGG/ADJ FL KIDS	1
2-6	881202	SIDE GUARD REC W/CLMP (M2/M3)	
2,4-6	881200	SIDE GRD/KIDZ ARM REC W/CLMP (3D)	
1A,2-6	901115	SIDE GUARD ASSM STD M3/M2	
1-6	901116	SIDE GUARD ASSM KIDS M3/M2	
1,2,4-6	901111	PLASTIC SIDE GUARD KDS ASSM (3D)	
1A,2,4-6	901110	PLASTIC SIDE GUARD STD ASSM (3D)	

PLASTIC SIDE GUARD (QUICKIE ALL COURT/MATCHPOINT)

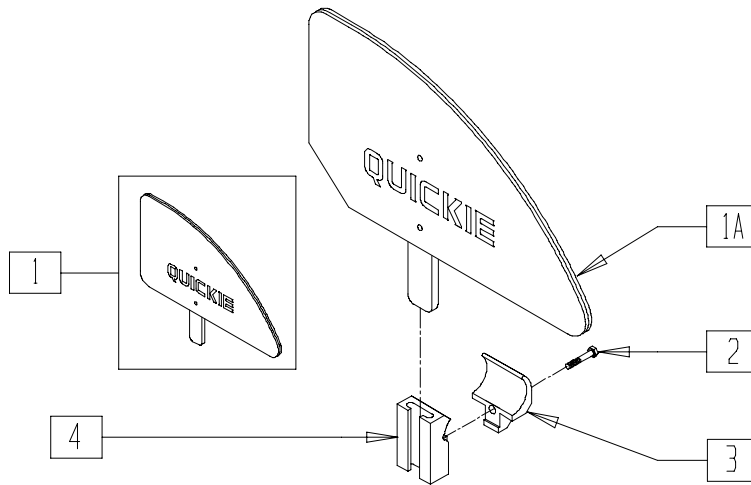

MEASURING GUIDE

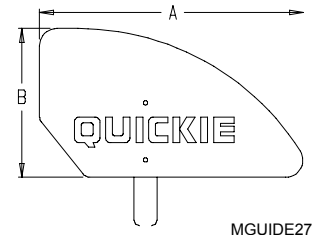

MGUIDE27

	"A"	"B"
ADULT	12 3/4"	8 5/8"
KIDS	11 1/2"	6 1/2"

Drawing shows right side


ALC_SDGD

ITEM	NUMBER	DESCRIPTION	QTY
1	881150, 881151	PLASTIC SIDE GUARD KIDS (EA)	1
1A	881152, 881153	PLASTIC SIDE GUARD STD (EA)	1
2	226142	1/4-20 x 1 SOC HD C/S	1

ITEM	NUMBER	DESCRIPTION	QTY
3	550425	SIDE GUARD CLAMP BLK QTNT	1
4	550428	SIDE GUARD RECEIVER BLK QTNT	1
1-4	901093	PLST SIDE GUARD KIDS AC (PR)	
1A,2-4	901092	PLST SIDE GUARD STD AC (PR)	

NOTE: WHEN ORDERING PLASTIC SIDEGUARDS MATCHPOINT CHAIR SHOULD HAVE BACK RELEASE BRACKET ASSEMBLY 920568 OR 920569 FOUND ON PAGES 2.6 AND 2.7 OF THIS PARTS MANUAL.

SIDE GUARDS (QUICKIE HEAT)

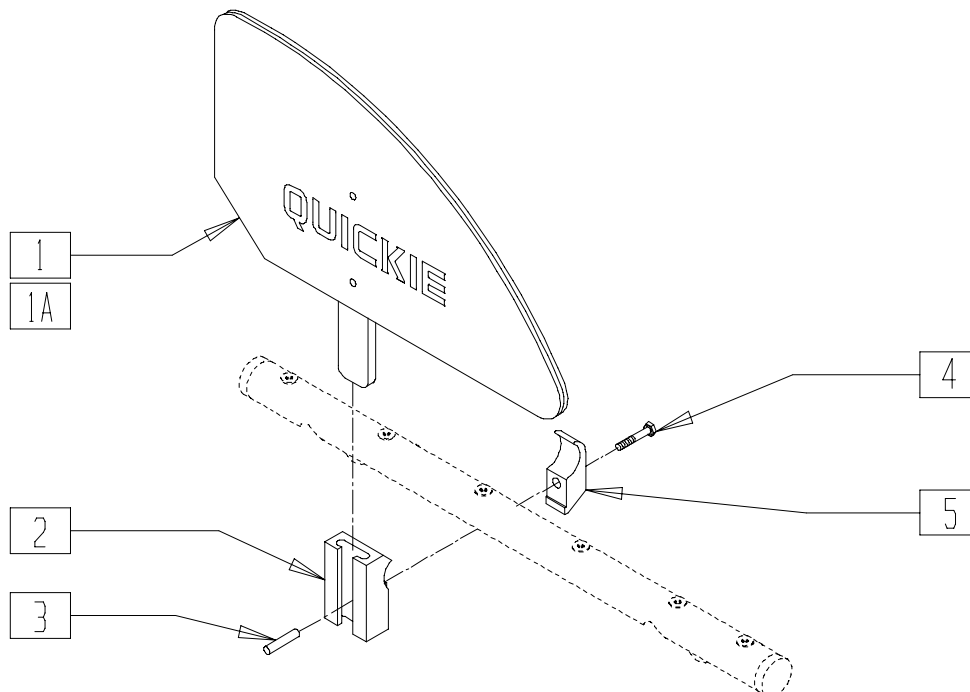


RG000X13

ITEM	NUMBER	DESCRIPTION	QTY
1	881150, 881151	PLASTIC SIDE GUARD KDS (EA)	1
1A	881152, 881153	PLASTIC SIDE GUARD STD (EA)	1
2	550326	QC SIDE GUARD REC	1


ITEM	NUMBER	DESCRIPTION	QTY
3	341130	1/4 x 1 1/8 ROLL PIN SS	1
4	226112	1/4-28 x 1 SOC C/S BLK	1
5	550723	HB CLAMP 3/4" NO NOTCH ST/DT	1
1A,2-4	901130	SIDE GUARD ASSM STD HQR	
1-5	901131	SIDE GUARD ASSM KIDS HQR	

SPOKE GUARDS

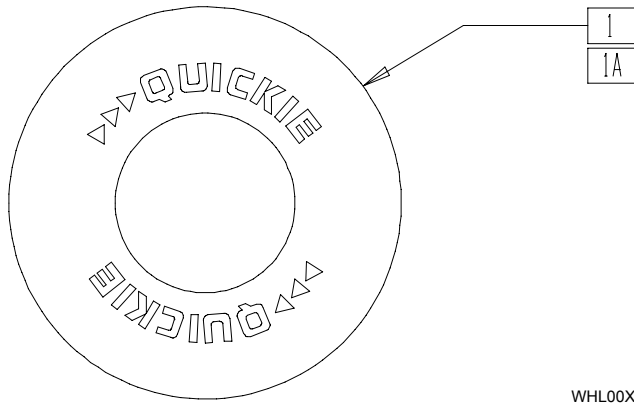

WHL00X24

MEASURING GUIDE

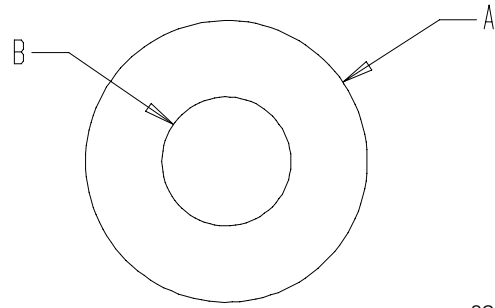


SG1

ITEM	NUMBER	DESCRIPTION	QTY
1	902054, 902056	QUICKIE SPOKE GUARDS CLR (PR)	1
1A	902001, 902011, 902021, 902025, 902037, 902038	QUICKIE SPOKE GUARDS BLK (PR)	1

SPOKE GUARD	NUMBER	DIA.A	DIA.B
SPOKE GUARDS 24" QP CLEAR	902054	20 1/2"	10 1/4"
SPOKE GUARDS 26" QP CLEAR	902056	23 1/2"	10 1/4"
SPOKE GUARDS 20" QUICKIE BLK	902001	15 7/8"	8 1/4"
SPOKE GUARDS 22" QUICKIE BLK	902011	18 1/8"	8 1/4"
SPOKE GRDS 24" SCR MT BLK (24" STD)	902021	19 1/2"	10 3/8"
SPOKE GRDS 24" TAB MT BLK (24" HT)	902025	19 5/8"	10 1/4"
SPOKE GRDS 26" TAB MT BLK (26" STD)	902037	22"	10 1/4"
SPOKE GUARDS 26" HT BLK (26" HT)	902038	23 1/4"	10 3/8"

SPOKE GUARDS (SPORT)

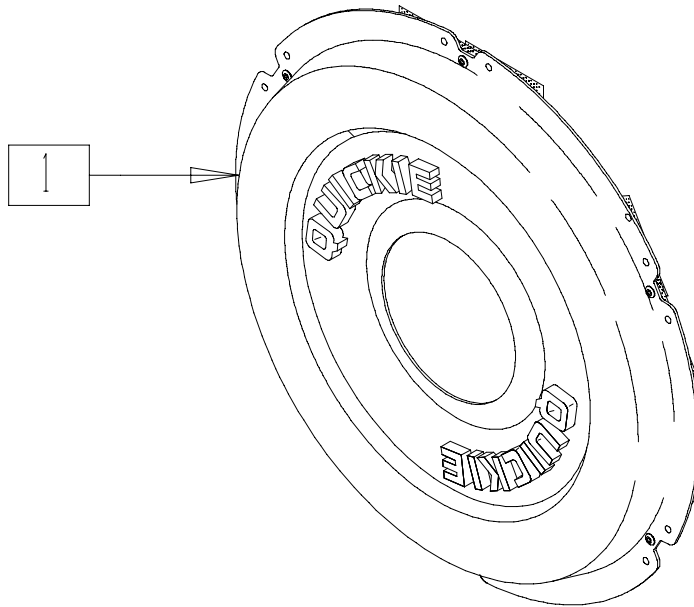

SPKGUARD

ITEM	NUMBER	DESCRIPTION	QTY
1	902072-902077	SPK GRD 24" SPRT HI PRFL W/TIE	1


SPOKE GUARD (SPORT) REFERENCE GUIDE

Wheel/ Hand Rim	24" Sport Hi Profile 902072	24" Sport Low Profile 902073	24" H.T. Sport Hi Profile 902076	24" H.T. Sport Low Profile 902077
24" STD RIM (540mm) ANODIZED or PAINTED HR REG or LONG TAB	◆	◆		
24" STD RIM (540mm) ANODIZED or PAINTED HR SHORT TAB		◆		
24" STD RIM (540mm) PLASTIC COATED HR REG or LONG TAB	◆	◆		
24" STD RIM (540mm) PLASTIC COATED HR SHORT TAB		◆		
24" STD RIM (540mm) ANY STYLE HR SCREW TAB	◆◆	◆◆		
24" H.T. RIM (559mm) ANODIZED or PAINTED HR REG or LONG TAB			◆	◆
24" H.T. RIM (559mm) ANODIZED or PAINTED HR SHORT TAB				◆

Wheel/ Hand Rim	24" Sport Hi Profile 902072	24" Sport Low Profile 902073	24" H.T. Sport Hi Profile 902076	24" H.T. Sport Low Profile 902077
24" H.T. RIM (559) PLASTIC COATED HR REG or LONG TAB			◆	◆
24" H.T. RIM (559) PLASTIC COATED HR SHORT TAB				◆
26" STD RIM (590) ANY STYLE HR REG or LONG TAB			◆◆◆	◆◆◆
26" STD RIM (590) ANY STYLE HR SHORT TAB				◆◆◆
26" STD RIM (590) ANY STYLE HR SCREW TAB			◆◆◆	◆◆◆
26" H.T. RIM (700C) ANY STYLE HR REG or LONG TAB			◆◆◆	◆◆◆
26" H.T. RIM (700C) ANY STYLE HR SHORT TAB			◆◆◆	◆◆◆


- ◆ RECOMMENED BEST FIT
- ◆◆ EXTRA EFFORT TO INSTALL; DAMAGE TO SPOKE GUARD OR WHEEL MAY OCCUR, BUT THEY WILL FIT AND FUNCTION O.K.
- ◆◆◆ EXTRA LARGE HANDRIM/WHEEL ASSEMBLY; SPOKE GUARDS NOT DESIGNED FOR THIS SIZE, BUT THEY WILL FIT AND FUNCTION O.K.

TOOL KIT (QUICKIE CYCLONE)

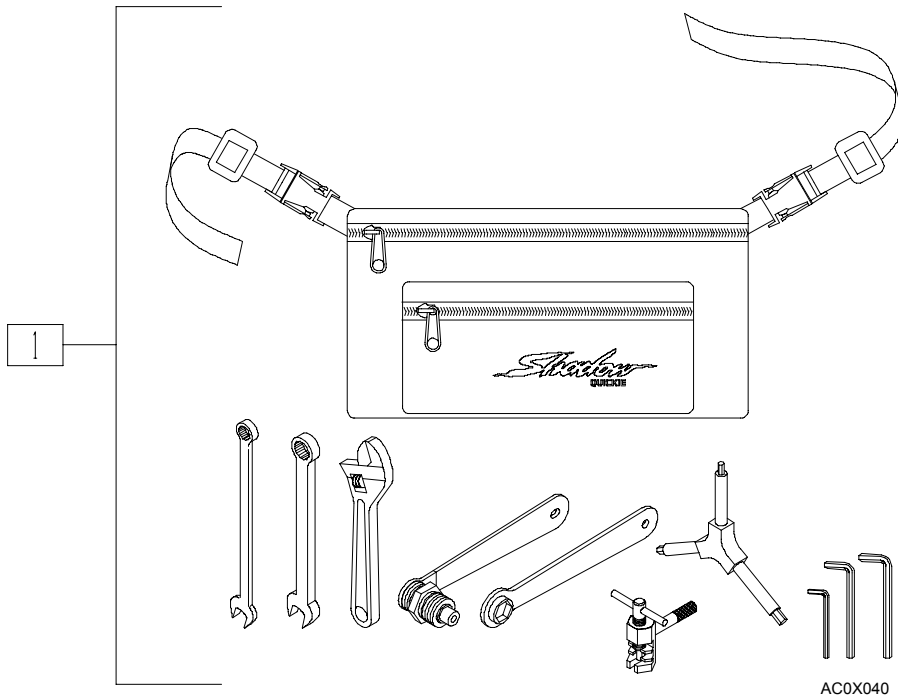


ITEM	NUMBER	DESCRIPTION	QTY
1	349888	TOOL KIT CYC	1

TOOL KIT (QUICKIE MACH 2)

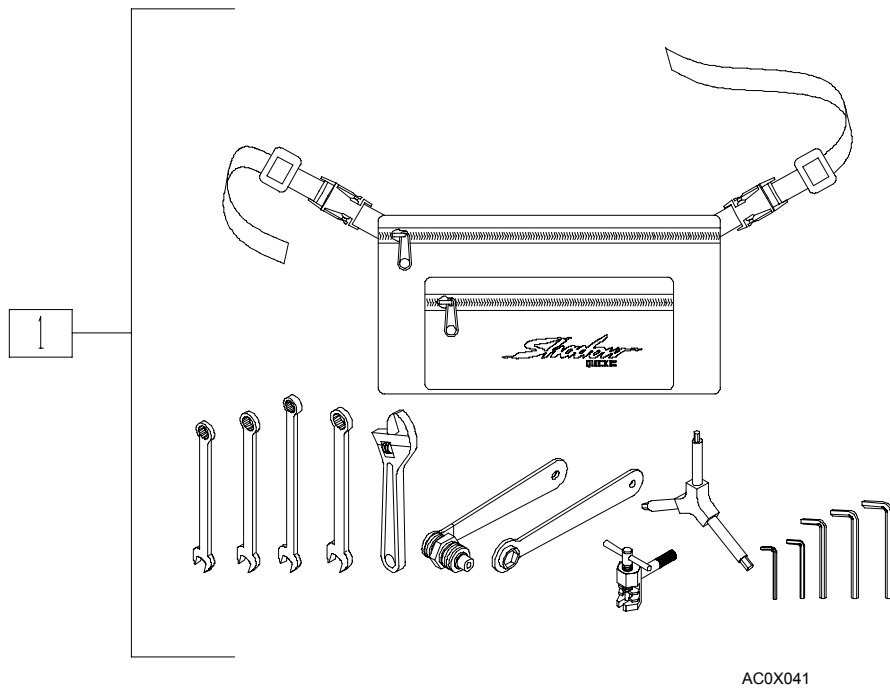

ITEM	NUMBER	DESCRIPTION	QTY
1	349995	TOOL KIT MACH 3/2	1

TOW BAR (QUICKIE MACH 2)

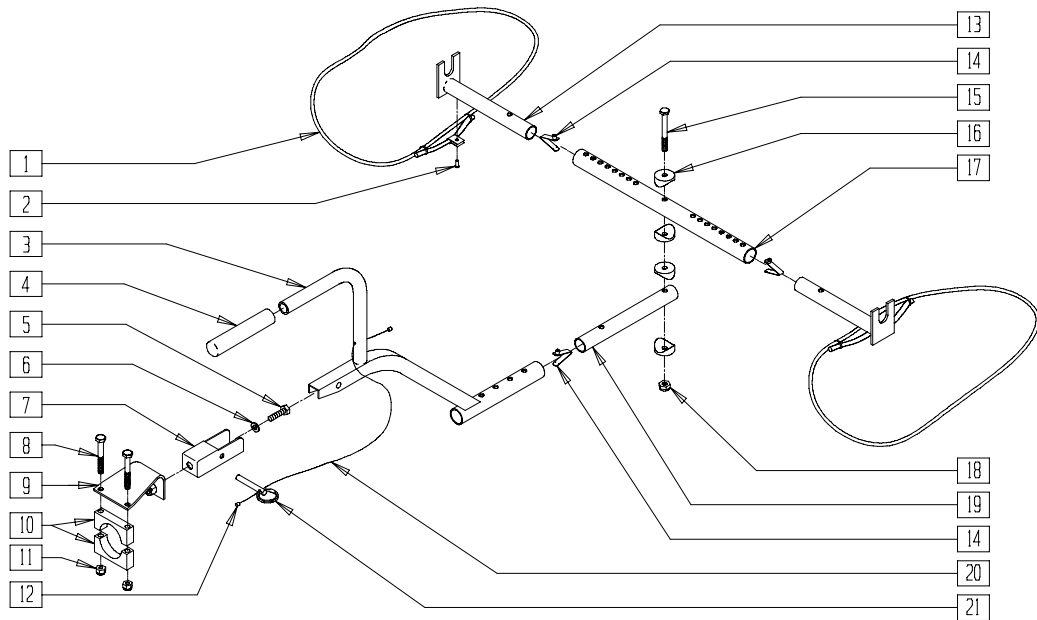


LC00X15A

ITEM	NUMBER	DESCRIPTION	QTY
1,2,13	881252	HITCH SUPPORT W/BUNGIE	
3,4,12,20,21	442309	TOW BAR DOWN TUBE W/FOAM	
5	222302	3/8 x 1 HEX HD BLT	1
6	263138	3/8 x 5/8 WASHER SS	1
7	520087	TRAILER HITCH FRAME REC	1
8	221180	1/4-28 x 2 1/2 HEX C/S	2
9	501150	TRAILER HITCH M2	1
10	501151	TRAILER HITCH CLAMP M2	2
11	201110	1/4-28 NYLOCK NUT	2

ITEM	NUMBER	DESCRIPTION	QTY
14	301100	1/4" SNAP BTTN SINGLE B-130	3
15	226308	1/4-20 x 2 3/4 HEX C/S SS	1
16	561003	3/4" SADDLE THIN	4
17	520088	TRAILER HITCH CROSS PIECE	1
18	201136	1/4-20 NYLOCK NUT HIGH	1
19	520085	TOW BAR EXT TUBE ANO	1
8-12	920395	TOW BAR ADAPTER M2	
14,19	881253	TOW BAR EXT W/SNAP BTTN	
1-7,12-21	442310	TOW BAR COMPLETE M3	
1-21	442311	TOW BAR COMPLETE M2	

WATER BOTTLE (QUICKIE CYCLONE/KIDZ BIKE/MACH 2)

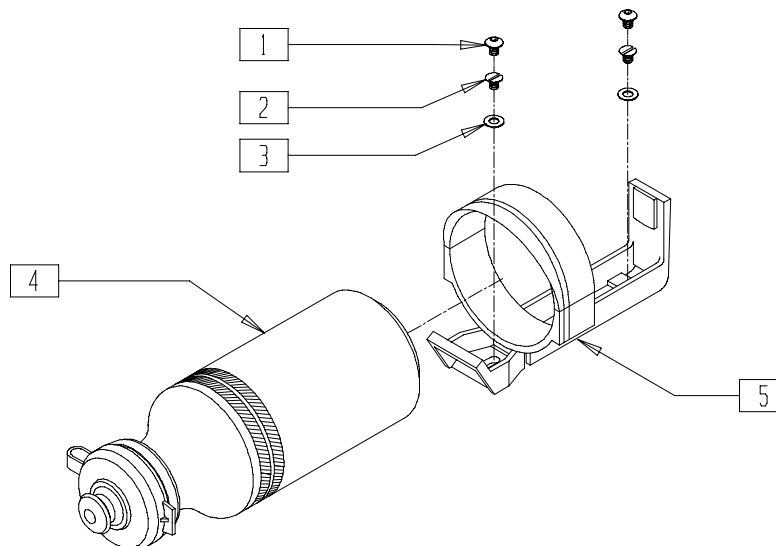

AC0X036

ITEM	NUMBER	DESCRIPTION	QTY
1	226042	10-24 x 1/2 BH C/S SS	2
2	281122	10-32 x 3/4 PAN PHIL (CYCLONE)	2
3	262099	3/16 ALUM WASHER BACKUP	2
4	929106	QUICKIE WATER BOTTLE	1

ITEM	NUMBER	DESCRIPTION	QTY
5	444129	LOCKJAW BOTTLE CAGE	1
2-5	920396	WATER BOTTLE AND CAGE ASSM C1/M2	
1,3-5	921396	WATER BOTTLE AND CAGE ASSM M3	

