



PART LIST


Revised June, 2001

IMPORTANT: Read the instruction manual before installing, operating, or servicing this system


ERGOLIFT®

FLOOR LIFT

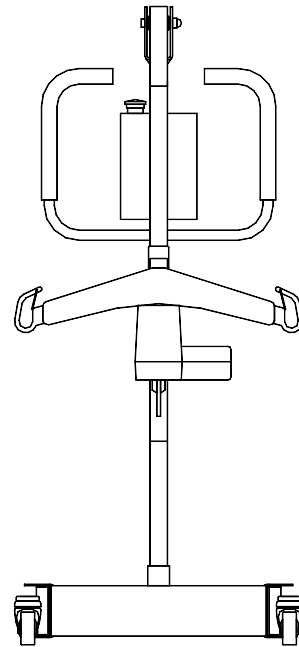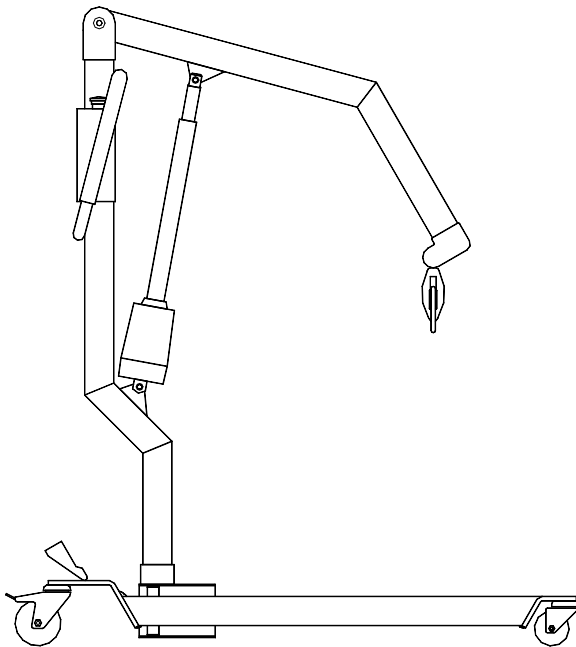

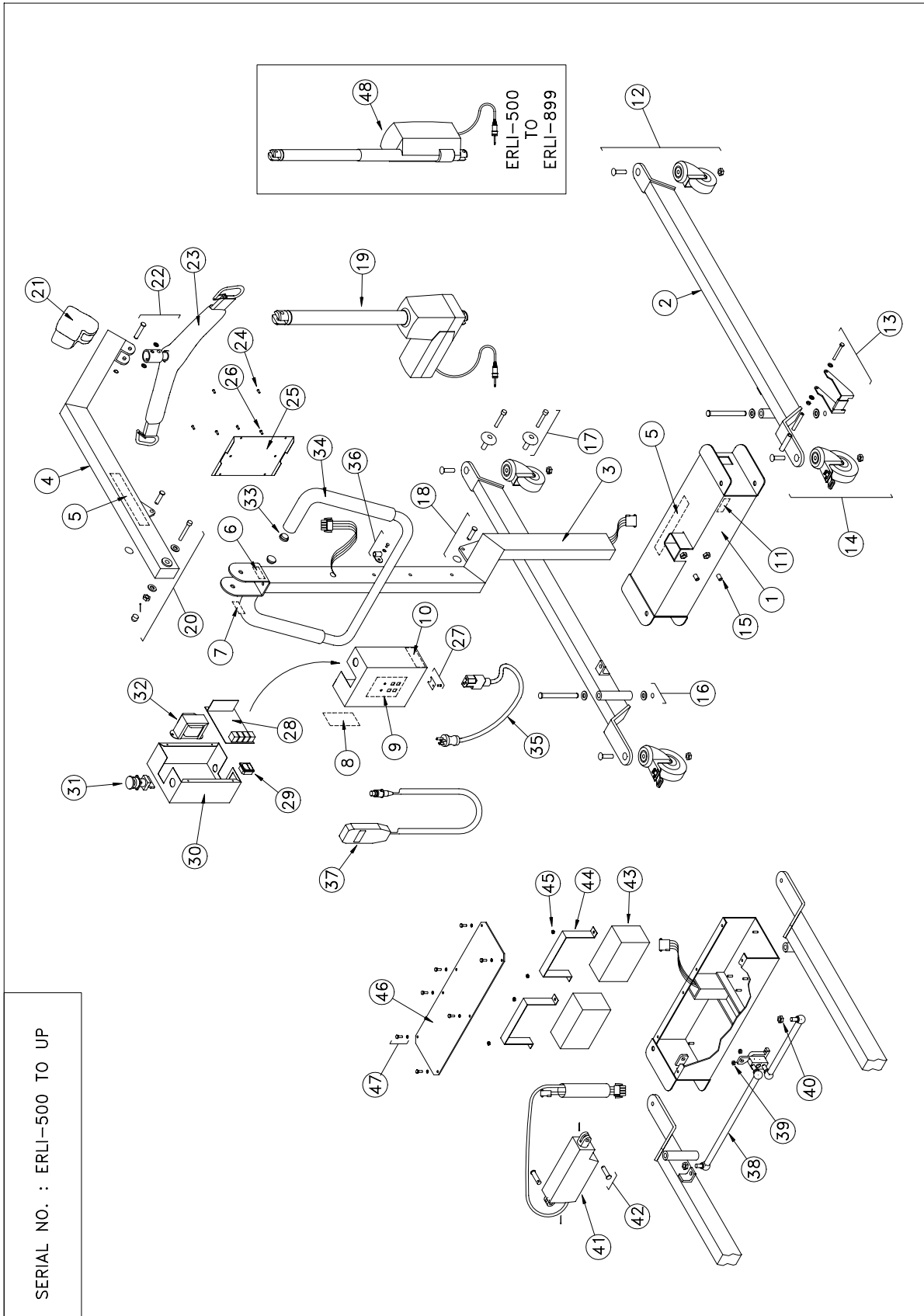

SERIAL NO. : ERLI-500 TO UP

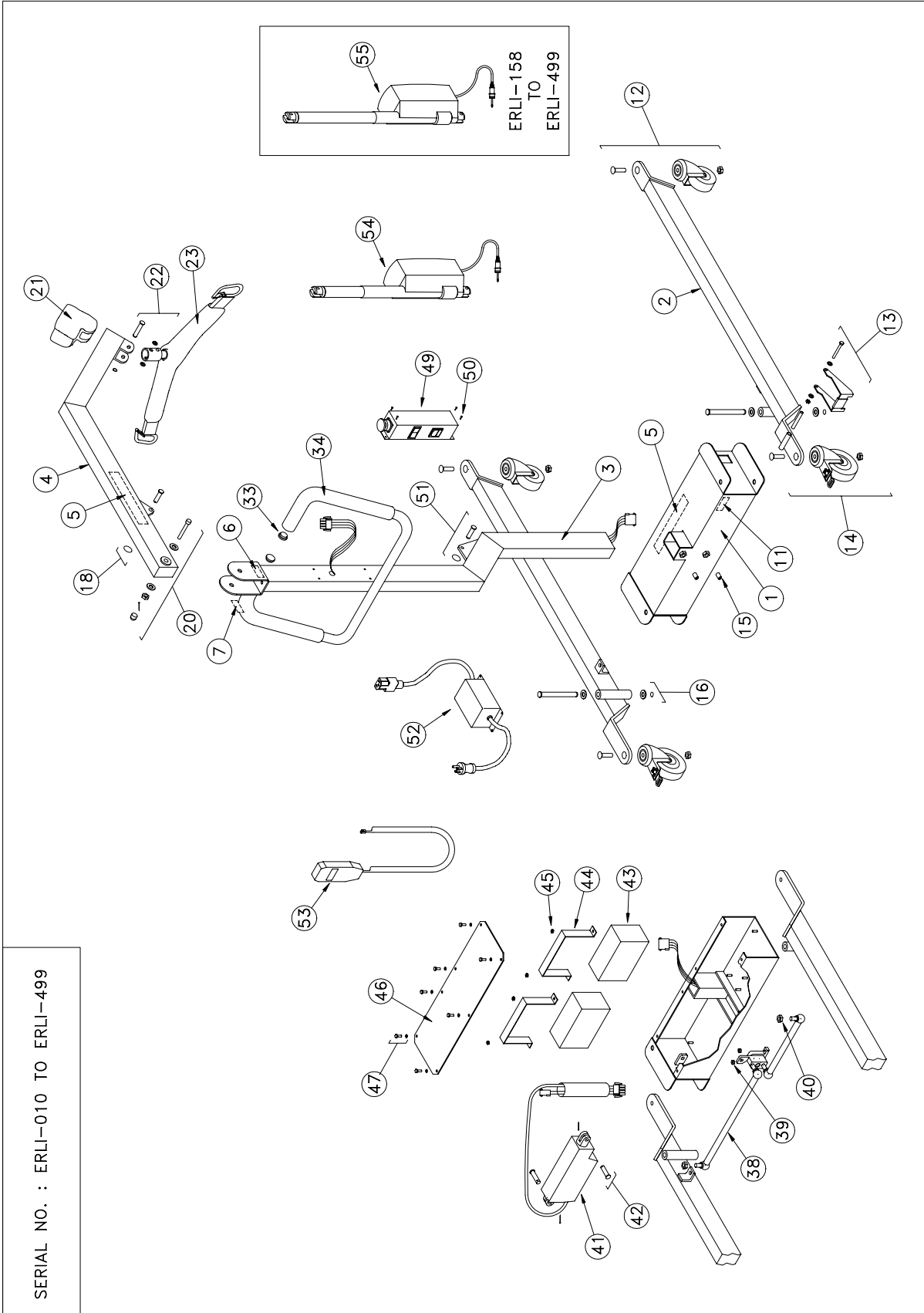

<u>ITEM NO.</u>	<u>QTY.</u>	<u>PART NO.</u>	<u>DESCRIPTION</u>
1	1	M6040	BASE
2-1	1	A6020	RIGHT LEG
2-2	1	A6030	LEFT LEG
3	1	M6010	ERGOLIFT MAST
4	1	M6000	ERGOLIFT BOOM
5	3	P6000	ERGOLIFT LABEL
6	1	P3000	BHM LABEL
7	1	P3005	BHM SERVICE LABEL
8	1	P3509	POSITION SLING LABEL
9	1	P6700	CONTROL BOX LABEL
10	1	P6710	CONTROL BOX CSA LABEL
11	1	P3020	SERIAL NO. LABEL
12-1	2	000.06000	3" FRONT CASTER
12-2	2	000.04250	M10 X 40 FLAT SCREW
12-3	2	000.04410	M10 LOCKNUT
13-1	1	A6060	STEERING LOCKING DEVICE
13-2	1	000.03230	M6 X 75 HEX SCREW
13-3	2	000.03890	8mm NYLON WASHER
13-4	1	000.03410	M6 LOCKNUT
13-5	1	000.03185	SPACER BUTTITE 6 X 60
14-1	2	000.06010	4" REAR LOCK CASTER
14-2	2	000.04250	M10 X 40 FLAT SCREW
14-3	2	000.04410	M10 LOCKNUT
15	2	000.04050	M10 X 16 SETSCREW
16-1	2	M6070	ERGOLIFT BASE SHAFT
16-2	4	000.04900	12mm NYLON WASHER, 2.5T
16-3	2	000.00852	SNAP RING 1/2"
17-1	2	M6510	CORD KNOB
17-2	2	000.03730	M8 X 50 SOCKET SCREW
18-1	2	M6150	10 X 38.5 CLEVIS PIN
18-2	2	000.00921	SPLIT RING
19	1	E6520	ACTUATOR LA-34-350
20-1	1	000.04600	M12 X 60 SHOULDER
20-2	2	000.04900	12mm NYLON WASHER
20-3	1	000.04410	M10 LOCKNUT
20-4	1	000.01800	2 X 20 CLEVIS PIN
20-5	1	000.04420	LOCKNUT CAP

21	1	A6200	END OF BOOM PADDING
22-1	1	M6151	11 X 52 CLEVIS PIN
22-2	2	000.00980	1/2" NYLON FLAT WASHER, 1/16T
22-3	1	000.04515	11mm RETAINING RING
23	1	A5465	CARRY BAR
24	4	000.00605	#6 X 3/8 TAPING SCREW
25	1	E6710	BOX COVER
26	2	000.00890	3/16 0-1/8 RIVET
27-1	1	E6715	LOCK JACK 1/4
27-2	1	000.00605	#6 X 3/8 TAPING SCREW
28	1	E6760-4F	ERGOLIFT-2 CIRCUIT
29	1	C5CBUL/MFC1	ETA CLIP CONECTOR 1mm
30	1	A6700	2 SPEEDS CONTROL BOX
31	1	C6P22M	22mm PUSHBUTTON
32	1	E7510	TRANSFORMER 120 24 50 VA
33	2	M6025	PLASTIC CAP
34	2	M6075	FOAM HANDLE
35	1	E7525	POWER CORD N. AMERICA
36-1	1	E0296	5/16 CABLE CLAMP
36-2	1	000.00605	#6 X 3/8 TAPING SCREW
36-3	1	000.00090	#6 STARWASHERS
37	1	A6800	6 BUTTONS HANDSET
38	1	A6110	ROD BASE SPREADING ASSEMBLY
39	2	000.02800	M5 LOCKNUT
40	2	000.04410	M10 LOCKNUT
41	1	A0072	ACTUATOR LA-12-70
42-1	2	000.04150	10 X 28 CLEVIS PIN
42-2	2	000.00860	3/32 X 3/4 COTTER PIN
43	2	A0150	BATTERY SET
44	2	M6050	BATTERY SUPPORT
45	4	000.02800	M5 LOCKNUTS
46	1	M6045	BASE COVER
47-1	7	000.02550	M5 X 12 SCREW
47-2	7	000.02900	M5 LOCKWASHER
48	1	E6305	ACTUATOR LA-32-350
49	1	A6000	ERGOLIFT-1 CONTROL BOX
50	4	000.00605	#6 X 3/8 TAPING SCREW
51-1	1	000.00905	7/16 X 1 21/32 CLEVIS PIN
51-2	1	000.00921	SPLIT RING
52	1	BHM-2450	EXTERNAL CHARGER

53	1	E0090	HANDSET 2 BUTTONS – TELEPHONE
54	1	E0055	ACTUATOR LA-32-300
55	1	E6300	ACTUATOR LA-32-350