



The Triton *Pool Lift*

The Triton Lift Manual



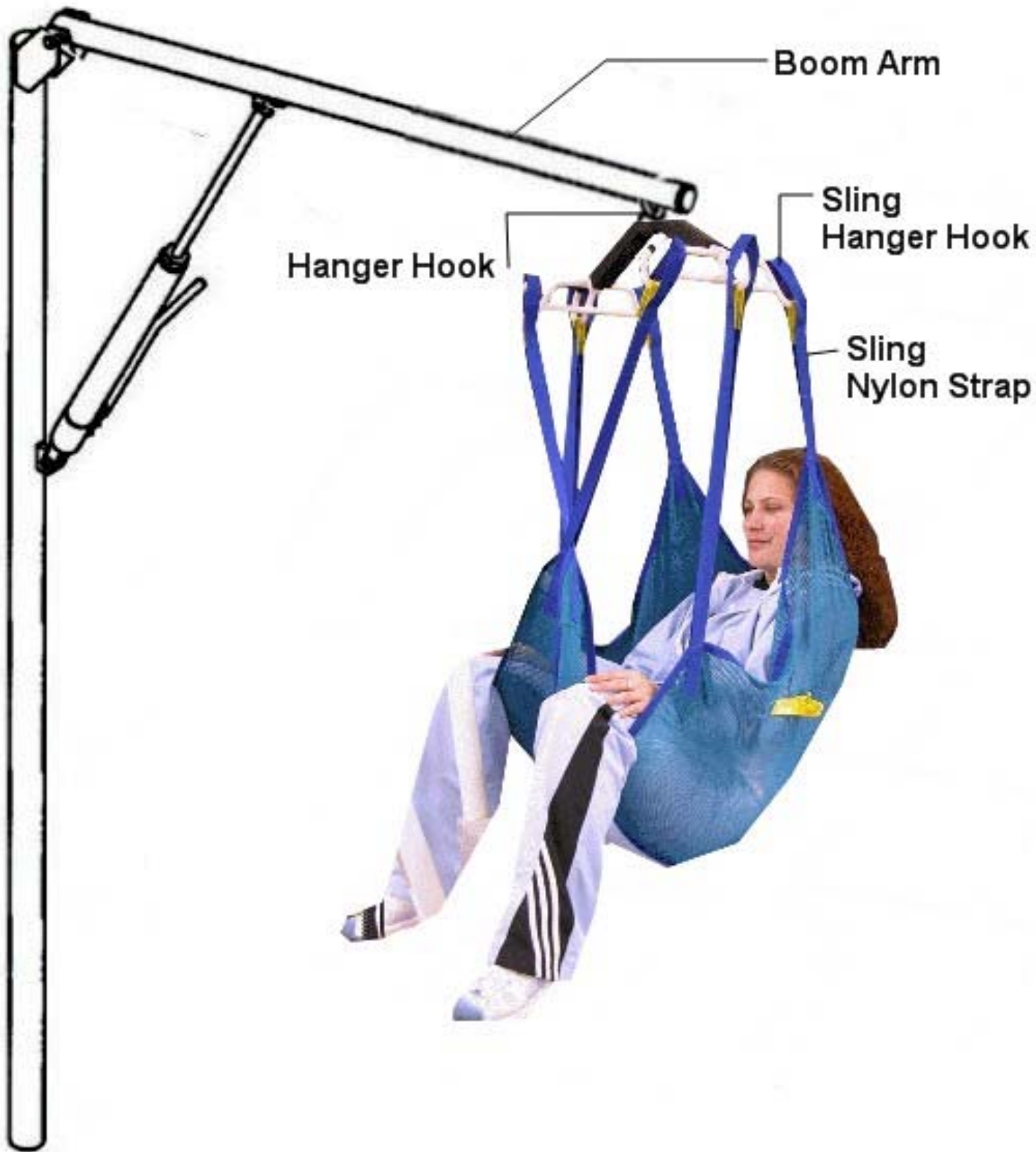

Parts


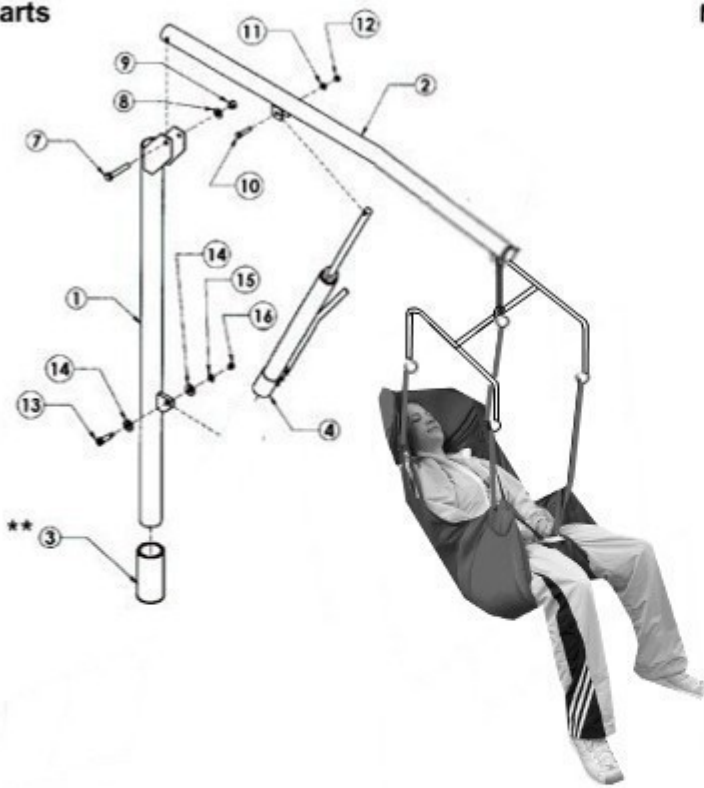


Num	Number	Description	Quantity	Part #
1	1	Mast Assembly	1	3700301
2	2	Boom Arm Assembly	1	3700302
3	3	Socket Anchor	1	3700303
3a	3a	Triton Anchor Cap	1	3700304
4	4	Hydraulic Ram Assembly	1	3700306
5	5	Hanger Assembly	1	3700307
6	6	Mesh Sling	1	3700308
HAF HARDWARE KIT (part #28500.08-00) includes:				
7	7	Hex Head Bolt 1/2" 13x3"	1	3700309
8	8	Flat Washer 1/2"	1	3700310
9	9	Nylon Nut 1/2"	1	3700311
10	10	Hex H/Bolt 3/8" 16x1-3/4"	1	3700312
11	11	Lock Washer 3/8"	1	3700313
12	12	Hex Nut 3/8" - 16	1	3700314
13	13	Sholder Bolt 1/2" 1-1/4"	1	3700315
14	14	Plastic Washer 1/2"	3	3700316
15	15	Flat Washer 3/8"	1	3700317
16	16	Nyloyoc Nut 3/8" - 16	1	3700318



Nylo Nylon Sling (part # 7743-778907) Includes:

20	20	Small Quick Dry Quick Fit Sling
20	20	Medium Quick Dry Quick Fit Sling
20	20	Large Quick Dry Quick Fit Sling

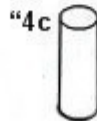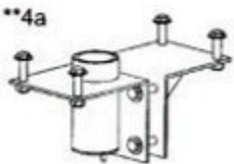

****Options:**

4a	4a	Wood Deck Anchor	1	3700319
4b	4b	Surface Mount Anchor	1	3700320
4c	4c	Triton Anchor Adaptor	1	3700321

The parts list above is a complete list of all parts used to build the Triton. All parts are readily available however normally 'Not' needed.


Hanger Assembly: The Triton offers a standard 2 Point Carry Bar.

Surface Mount: The surface mount is designed strictly for wood surface such as a patio, deck, or a wood floor. Note: It is very important to be sure you drive the lag bolts directly in the CENTER of the Floor Joist. It is very important that you load sets each lift lifting at least 400 lbs to be sure that there is no weak points to your installation.


Pool Deck Worksheet: This worksheet is not a requirement to order. This worksheet can be used as a helpful guideline to demonstrate the placement of your anchor in relation to the pool wall. This worksheet can also be used to determine anchor placement with an above ground Jacuzzi, Hot tub or Pool.

Tech Support: Technical support is available to answer any questions you may have regarding the Triton and or the installation of the Triton at (866) 465-4387 or email info@worldwideseating.com

Pool Deck Profile Worksheet

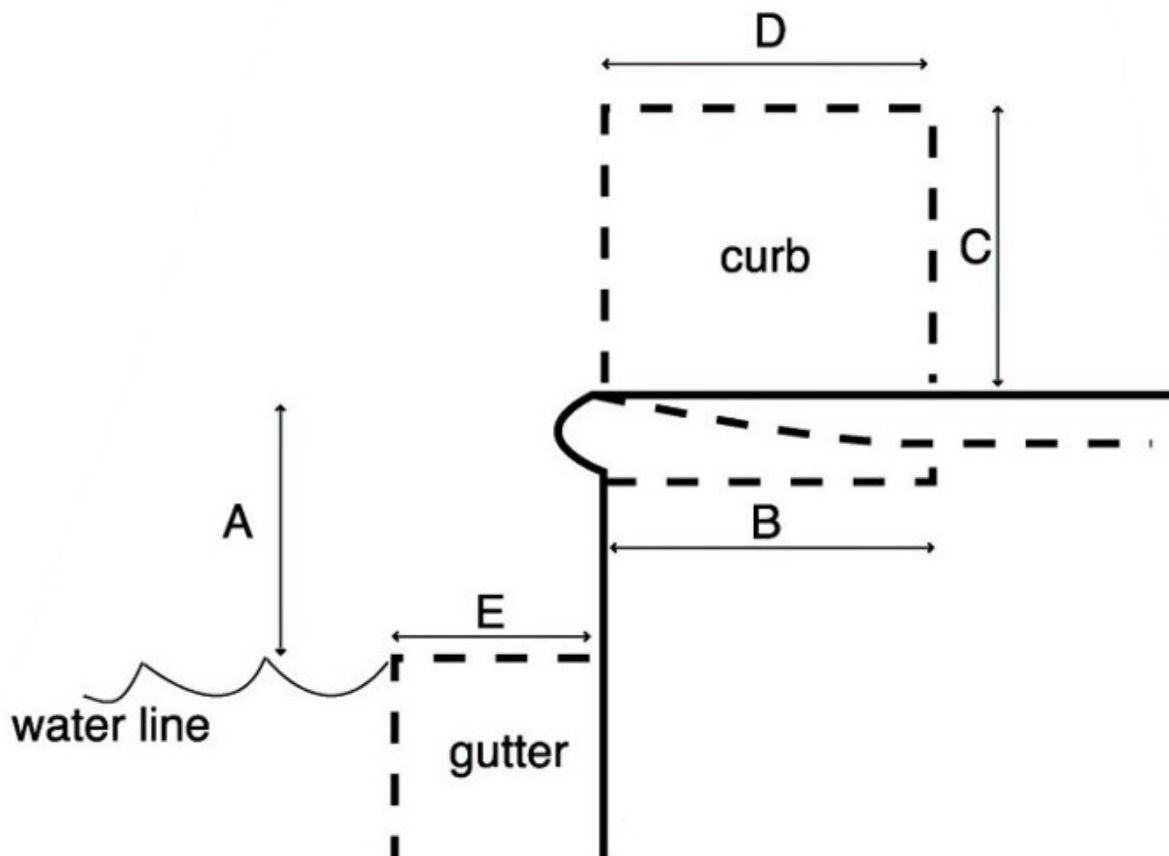

- A. Distance from pool deck to water line – Mandatory. _____
- B. For curbed (bullnose) coping – distance from level portion of pool to edge of pool – if applicable. _____
- C. Height of curb – if applicable. _____
- D. Width of curb – if applicable. _____
- E. Width of gutter – if applicable. _____

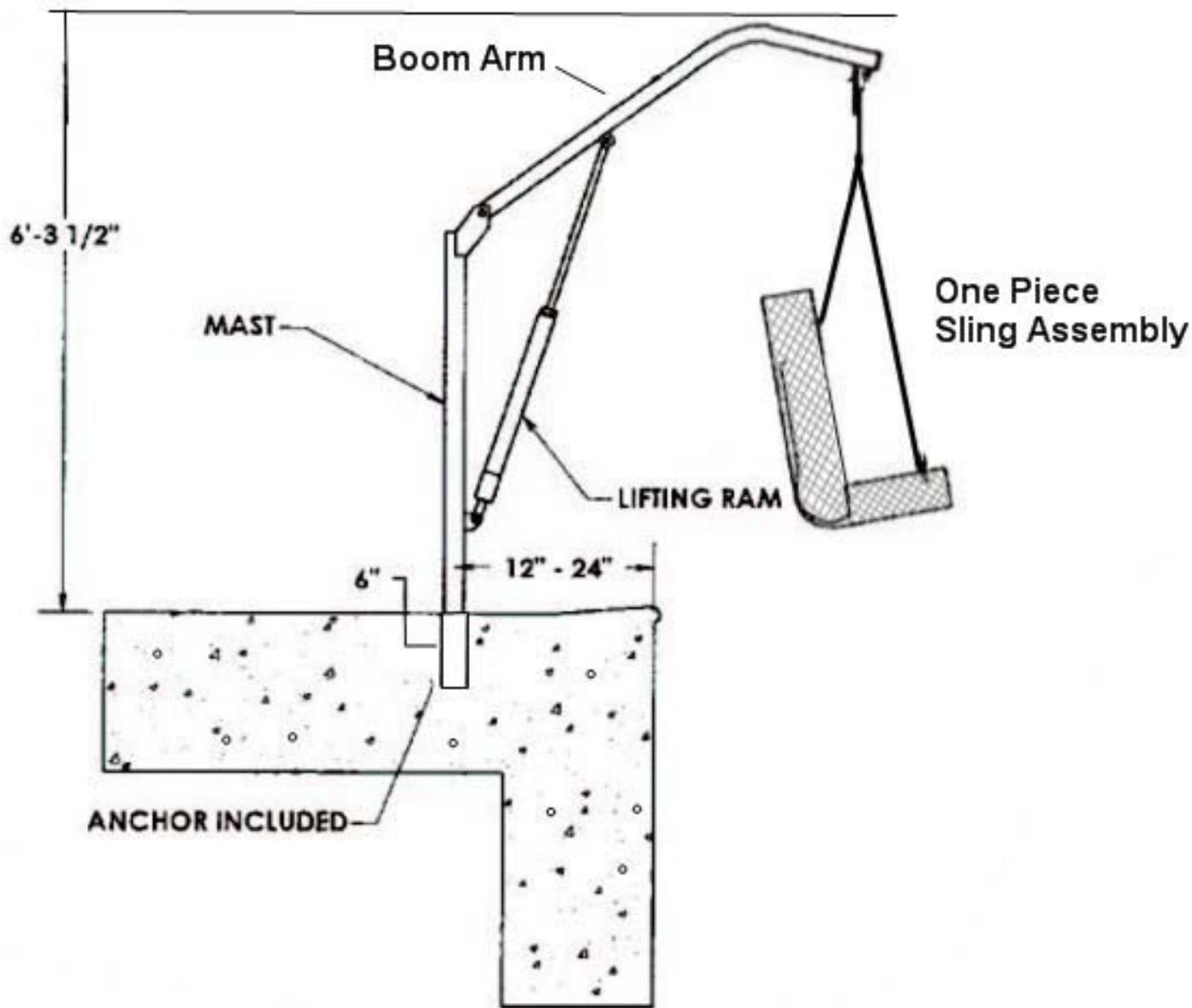

Installing The Anchor:

Concrete: There are two separate ways to installing the Triton concrete anchor. First you may drill a hole in the deck of the pool 3.25" inches as shown above. This application would then allow you to simply slide the Anchor right into the hole you have drilled.

Wood Surface Mount: This application is used with wood floors. This installation requires this anchor to be bolted to the floor joist under the floor.